



## Your eye rinsing solution with hyaluronic acid

- ✓ First hyaluronic acid eye rinsing solution
- ✓ Restores immediate ocular comfort
- ✓ Large volume of solution (20 ml)
- ✓ Adaptable flow (drops or intensive irrigation )
- ✓ Sterile for 3 months after opening
- ✓ Cost effective



Distributor

TRB CHEMEDICA INTERNATIONAL SA - Geneva - Switzerland  
Tel: + 41 22 703 49 00 - Fax: + 41 22 703 49 01  
ophthalmology@trbchemedica.com - www.trbchemedica.com



DUOVISUAL - GENEVA - CH - 9427 - 03/13 VISM-14a

# VISMED® WASH ...

## ...the first eye rinsing solution with hyaluronic acid



For irrigation, rewetting and lubrication



# Washes away irritants from your eyes...

## VISMED® WASH mildly and gently washes away irritants such as:

- › Cellular debris
- › Mucus filaments
- › Allergens
- › Small foreign bodies
- › Chemical agents



from the eye or the surface of in-eye contact lenses.

**VISMED® WASH** can also be used in case of ocular surface disorders, blepharitis, conjunctivitis, keratitis.

**VISMED® WASH** is the first eye rinsing solution containing 0,075% hyaluronic acid, a biopolymer naturally present in the eye that provides prolonged wetting of the ocular surface with its ability to retain water.

**VISMED® WASH** is available in a squeezable bottle containing 20 ml of solution. The outflow can easily be adjusted by applying pressure on the body of the container.

Depending on the level of ocular contamination, the total content of one bottle may be needed to flush away, for example, mild chemicals or foreign bodies whereas 5 drops per eye are often sufficient to remove cellular debris.

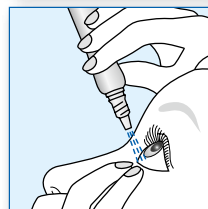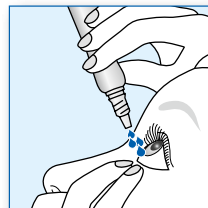

## Cellular debris


Some patients suffer from inflammation of the eyelid margin (blepharitis).

**VISMED® WASH** removes cellular debris and abnormal secretions from the eye.




## Allergens

**VISMED® WASH** flushes away allergens from the ocular surface and thus helps to decrease ocular inflammation.

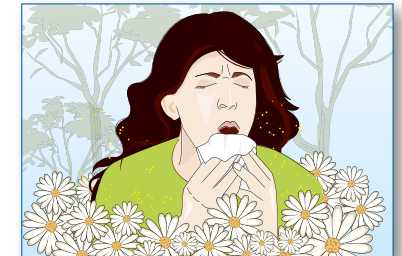

## Abrasive irritants

**VISMED® WASH** gently washes away abrasive irritants such as dust or sand that could damage the cornea, irritate eyelids and disturb the vision thereby restoring ocular comfort.

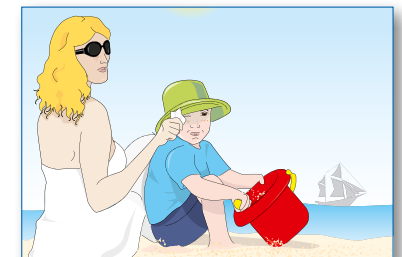

# ...and restores immediate ocular comfort.