



NEW

HIGH PROTECTION  
ANYTIME, ANYWHERE

NOW AVAILABLE  
IN PRESERVATIVE-FREE MULTIDOSE CONTAINER

- ✓ Ergonomic and intuitive delivery system
- ✓ 10 ml container
- ✓ Compact size design
- ✓ Preservative-free solution
- ✓ Sterile for up to 3 months after first use

And always:

- ✓ 0.3% sodium hyaluronate
- ✓ Hypotonic formulation
- ✓ Essential ions





## FOR SUSTAINED LUBRICATION OF THE EYES

### INSTRUCTIONS FOR USE

#### VISMED® GEL MULTI

Sodium hyaluronate from fermentation 0.3 %.  
Hydrogel for sustained lubrication of the eyes.  
Sterile, free from preservatives.

#### COMPOSITION

1 ml solution contains sodium hyaluronate 3.0 mg, sodium chloride, potassium chloride, disodium phosphate, sodium citrate, magnesium chloride, calcium chloride and water. The solution is hypotonic.

#### INDICATIONS

Symptoms or sensations of dry eye and/or signs of ocular surface damage, due to diseases such as superficial keratitis, Sjögren syndrome or primary dry eye syndrome. For sustained lubrication of the eyes in case of sensation of dryness, burning and ocular fatigue and other minor complaints of no pathological significance induced, for example, by dust, smoke, dry heat, air conditioning, wind, cold, extended computer screen, contact lens wear (rigid or soft) or by ophthalmological screening methods.

#### CONTRA-INDICATIONS

Individual hypersensitivity to any constituent of the product.

#### INTERACTIONS

Do not use VISMED® GEL MULTI at the same time as any drug or other product applied to the eye as it may modify their effects.

#### UNDESIRABLE EFFECTS

On very rare occasions transient disorders such as irritation of the conjunctiva, foreign body sensation, redness or burning sensation of the eye, short-term blurred vision may occur.

#### DOSAGE AND ADMINISTRATION

Tear off the tamper-evident seal before the first administration. Remove the protective cap. Tilt the head backwards and position the tip of the container above the eye to be treated. Move the lower eyelid down with the

forefinger of one hand. Press the container and one drop of VISMED® GEL MULTI will easily flow out. Unless otherwise recommended, instill 1 or 2 drops of VISMED® GEL MULTI onto the eye as often as needed. After blinking, the solution will disperse and form a transparent and long lasting coating on the surface of the eye. VISMED® GEL MULTI may also be used while wearing contact lenses (rigid or soft).

#### PRECAUTIONS

Do not touch the tip of the opened container and do not touch the surface of the eye with the tip of the container. Put the protective cap back on after using VISMED® GEL MULTI. Do not use VISMED® GEL MULTI if the container is damaged. VISMED® GEL MULTI can be used up to 3 months after first use. Any solution not used within 3 months after opening must be discarded, otherwise the sterility is no longer guaranteed and may be associated with an enhanced risk of infection. Store between 2 °C and 25 °C! Do not use VISMED® GEL MULTI after the expiry date indicated on the container and the folding box. If discomfort persists while using VISMED® GEL MULTI consult a physician. Keep out of reach of children.

#### CHARACTERISTICS AND MODE OF ACTION:

VISMED® GEL MULTI contains sodium hyaluronate, a natural polymer, which is also present in the structures of the human eye. The particular physical characteristics of sodium hyaluronate confer to VISMED® GEL MULTI its viscoelastic and water retaining properties. VISMED® GEL MULTI provides a stable coating on the surface of the eye which is only slowly eliminated by blinking. Therefore VISMED® GEL MULTI combines long lasting relief with maximum comfort. VISMED® GEL MULTI is well tolerated due to its unique composition. VISMED® GEL MULTI is free from preservatives.

Presentation: 10 ml solution in a multidose container.

Last revision date: March 2012