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VISMED® wash

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VISMED® wash

Sodium hyaluronate from fermentation 0.075 %.

Rinsing solution for the eye. For external use only.

Composition:

Each 1 ml of solution contains: sodium hyaluronate 0.75 mg, sodium chloride, disodium-hydrogenphosphate, sodium-dihydrogenphosphate, purified water, disodium edetate and polyhexanide as preservatives.

Intended use:

For irrigation, rewetting and lubrication of the eyes or contact lenses in case of discomfort or visual complaints related to various ocular surface disorders (including dry eye conditions). VISMED® wash can be particularly helpful to flush away excessive tear film debris, abnormal secretions, foreign bodies, allergens or chemical agents.

Contra-Indications:

Individual hypersensitivity to any constituent of the product.

Interactions:

Do not use VISMED® wash at the same time as any other drug or product applied to the eye since it may modify their effects.

Directions for use:

To open the multidose container for the first time, twist the screw cap firmly clockwise: a spike in the screw cap will pierce the tip of the container. Twist off the screw cap and position the tip of the container above the eye to be treated: press the sides of the container and VISMED® wash will easily flow out from the container. If not otherwise recommended, rinse the cornea and/or conjunctiva with VISMED® wash as often as needed: a few drops per eye are sufficient to remove debris whereas for flushing away chemicals or foreign bodies the content of one bottle may be needed. VISMED® wash may also be used while wearing contact lenses (rigid or soft) or scleral lenses. Once the container is opened, the solution may be used for up to three months.

Precautions for use:

Do not touch the tip of the opened container and do not touch the surface of the eye with the tip of the container. Do not use VISMED® wash if the container is damaged.

Store between 2 °C and 25 °C! Do not use VISMED® wash after the expiry date indicated on the package. If discomfort persists while using VISMED® wash or reappears after discontinuation of treatment, consult a physician.

Keep out of reach of children!

Characteristics and mode of action:

VISMED® wash contains sodium hyaluronate, a natural polymer, which is also present in the structures of the human eye. The particular physical characteristics of sodium hyaluronate confer to VISMED® wash its „viscoelastic“ and water retaining properties. VISMED® wash is well tolerated due to its unique composition. The particular composition of preservatives in VISMED® wash guarantees the sterility of the solution in the open container for a period of three months.

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